


# Handhabungshinweis

Urin-Monovette®

1

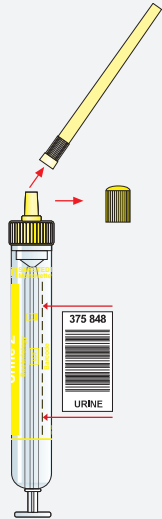


Bei Barcode- Etikettierung  
Etikett entlang der Linie  
kleben.

**Stopfen entfernen und  
aufbewahren!**

Die Saugspitze aufsetzen.

2

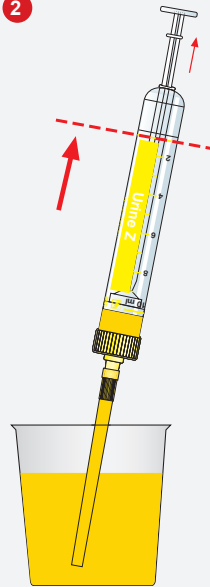


Die Saugspitze ins  
Gefäß eintauchen und  
die Urin-Monovette® bis  
zur **Basis-Linie** aufziehen.

3



Die Urin-Monovette®  
senkrecht halten und die  
Kolbenstange **bis zum  
Anschlag nach unten  
aufziehen**, um die Saug-  
spitze zu entleeren.

4

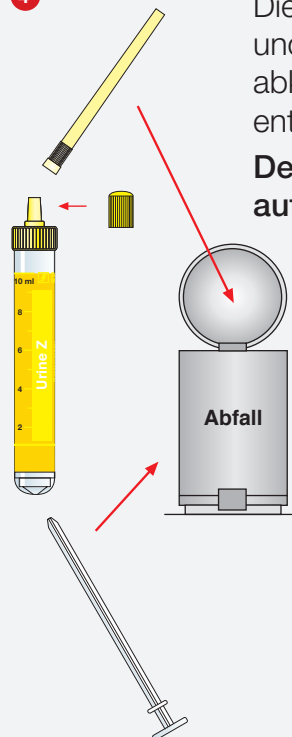

Die Saugspitze abziehen,  
und die Kolbenstange  
abknicken, beides  
entsorgen.

**Den Stopfen wieder  
aufsetzen.**


SARSTEDT AG & Co.  
Postfach 12 20 · D-51582 Nümbrecht  
Telefon +49 22 93 305-0  
Telefax +49 22 93 305-3450  
**☎ Service 0800 (Deutschland)**  
Telefon (0800) 0 83 305-0  
info@sarstedt.com  
www.sarstedt.com

 **SARSTEDT**

# Instruction for use

## Urine Monovette®

1

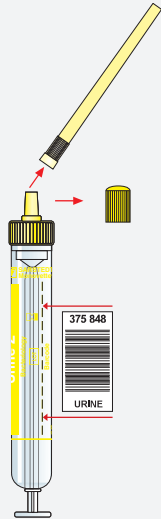


When using a barcode label, attach it along the printed line.

**Remove stopper and keep for later use!**


Attach aspiration tip.

2

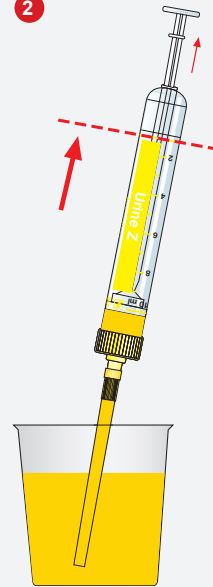


Insert the aspiration tip into the container and fill the Urine Monovette® up to the **base line**.

3

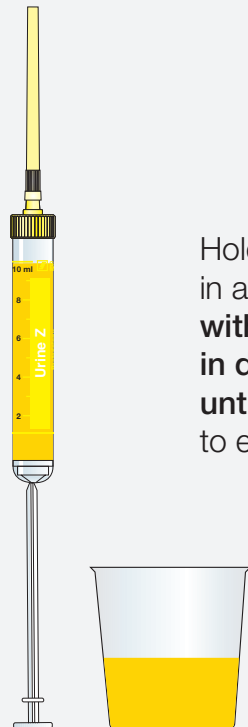

Hold the Urine Monovette® in a vertical position and **withdraw the piston rod in downward direction until it locks into its base** to empty the aspiration tip.

4

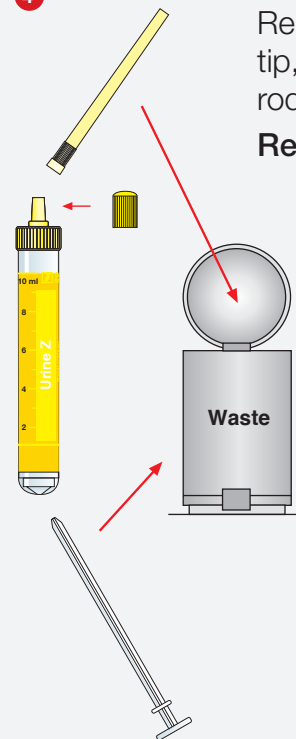

Remove the aspiration tip, break off the piston rod and discard both.

**Replace the stopper.**