

Urine Analysis

Urine Monovette® · Urine Monovette® with stabilizer
UriSet 24 · Urine Containers · Urine Cups



One disposable product with three functions

Hygienic urine collection

Urine Monovette® includes a **suction tip** for drawing urine from collection containers.



Alternatively, a **Luer needle** (not provided) can be attached for collecting catheter urine from the puncture sites of enclosed drainage systems.



Vessel for test strips

Test strip may be immersed into the filled Urine Monovette® for urine analysis.



Centrifuge tube for sedimentation

Urine Monovettes may be safely centrifuged for sedimentation. After centrifugation, sediment is easily accessed by decanting the supernatant and pushing the plunger to the tube opening.



For bacteriological urine analysis

- The Urine Monovette® with boric acid stabilizes microorganisms in urine while providing the advantages of a standard Urine Monovette®.
- When filled to 10 ml, urine is stabilized for up to 48 hours at room temperature.
- Extended viability reduces bottlenecks in the laboratory.
- Literature on bacteriostatic effect of boric acid on the most important causes of urinary tract infections is available on request.



Ordering information

Order No.	Description	Volume in ml	Ø in mm	Sales Unit
10.258	● Urine Monovette®*	8.5	15.3	512/case
10.258.020	● Urine-Monovette®, ind. wrapped, sterile*	8,5	15,3	500/case
10.252	● Urine Monovette®*	10	15.3	512/case
10.252.001	● Urine Monovette® with paper label*	10	15.3	512/case
10.252.020	● Urine Monovette®, ind. wrapped, sterile*	10	15.3	500/case
10.252.030	● Urine Monovette® white, for UV protection*	10	15.3	512/case
10.252.570	● Urine Monovette® kit (Urine Monovette®, suction tip, 100 ml container)	10	15.3	500/case
10.253	● Urine Monovette® with stabilizer*	10	15.3	512/case
10.253.001	● Urine Monovette® with stabilizer and paper label*	10	15.3	512/case
10.253.020	● Urine Monovette®, with stabilizer, ind. wrapped, sterile*	10	15.3	500/case
10.251	Suction tip for Urine Monovette®, separate			5,000/case

*incl. enclosed suction tip

UriSet 24 • Complete set for standardized and patient-friendly 24 hour urine collection



Complete set:

- 3 Liter urine container
- 500 ml collection and transfer cup
- Stabilizer – hydrochloric acid (9 ml 20%)
- Transport tube (30 ml transport tube or Urine Monovette®)

Order No.	Description	Sales Unit
77.578	UriSet 24 – 3 Liter urine container with 30 ml transport tube	30/case
77.578.252	UriSet 24 – 3 Liter urine container with Urine Monovette®	30/case
77.578.010	UriSet 24 – 3 Liter urine container with Urine Monovette® without hydrochloric acid stabilizer	30/case
75.1356	Collection and transfer cup – 500 ml	250/case
62.543.030	Transport tube – 30 ml, white for UV protection	500/case

Urine Containers • Range of sizes and options for urine collection



3 Liter urine container with assembled screw cap

- 3,000 ml filling volume
- Ergonomically moulded handle
- Extra large opening
- Transparent inspection strip

Order No.	Description	Sales Unit
77.580	2 Liter urine container for 24h urine with label	30/case
77.575.001	3 Liter urine container for 24h urine with label	30/case
84.1402.001	Label for urine collection containers, size 150 x 55 mm	300 (10x30/bag)/box; 1,800/case

Urine Cups • Various options with snap-on lid or screw cap



Order No. 75.565
125 ml Cup,
polypropylene, neutral,
500 pcs./case

Order No. 76.566
Snap-on lid,
2,000 pcs./case

Order No. 75.1355
125 ml Cup,
polypropylene, neutral,
1,000 pcs./case

Order No. 76.1477
Snap-on lid,
with decanter,
1,000 pcs./case

Order No. 76.1414
Snap-on lid,
without decanter,
1,000 pcs./case

Order No. 75.570
100 ml Cup,
polystyrene, white,
1,000 pcs./case

Order No. 76.571
Snap-on lid,
1,000 pcs./case

Order No. 75.560
250 ml Cup,
polystyrene, white,
500 pcs./case

Order No. 76.561
Snap-on lid,
500 pcs./case

Order No. 75.1354.001
100 ml Cup,
polypropylene, neutral,
with green screw cap,
polyethylene,
500 pcs./case

Order No. 75.9922.744
70 ml Cup,
with assembled screw cap,
500 pcs./case

Order No. 75.9922.745
with assembled screw
cap, with standard label,
500 pcs./case

Order No. 75.9922.721
with assembled screw
cap, sterile, ind. wrapped,
with standard label,
100 pcs./case

Order No. 75.563
100 ml Cup, polypropylene,
neutral, without cap,
500 pcs./case

Order No. 76.564
Screw cap, 500 pcs./case

Order No. 75.562.005
with assembled screw
cap, 200 pcs./case

Order No. 75.562.105
with assembled screw
cap, with safety label,
5 pcs./bag sterile,
200 pcs./case

Order No. 75.562.011
100 ml Cup brown,
screw cap enclosed,
250 pcs./case

Conical urine tubes are available on request.

SARSTEDT AG & Co.
P.O. Box 12 20
D-51582 Nümbrecht
Phone +49 2293 3050
Fax +49 2293 305-122
info@sarstedt.com
www.sarstedt.com