


UM 500 – The Urine Measuring System



- User-friendly precise urine monitoring system providing optimum hygienic safety

Hygienically safe

The integral drip chamber with bacteria-resistant ventilation and anti-reflux membrane valve combined with an additional **1** anti-reflux valve in the collection bag ensures it to be hygienically safe and prevents retrograde bacterial migration.

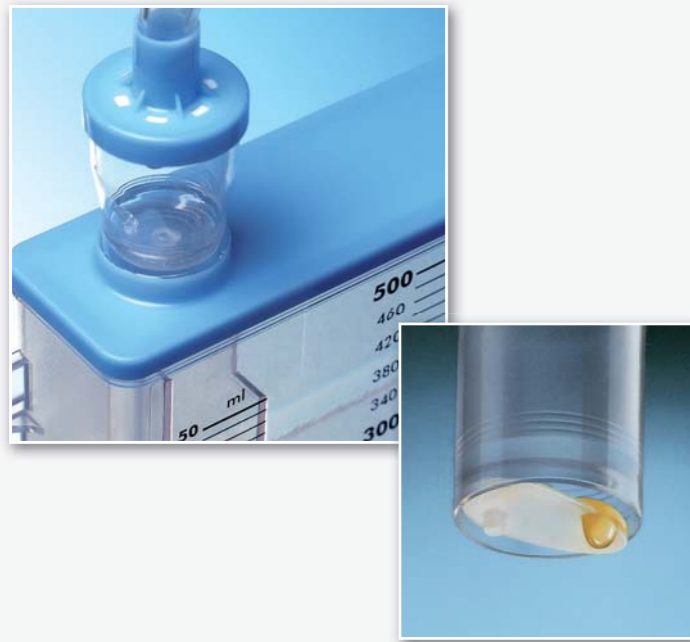

Individual attachment

Perforated elastic straps or specially designed side rail hooks (make Dräger) enable individual attachment of the UM 500 to any bed frame.



Rapid drainage


A special rotary valve and optimal pressure balancing throughout the system ensure rapid draining of the measuring chamber (bacteria-resistant ventilation of the drip chamber, measuring chamber and **2** collection bag).













Precise reading

The precise and easily readable graduations facilitate accurate output readings of even the most negligible volumes. The total measuring volume of 500 ml and the clear and concise graduations enable accurate balancing of large urine volumes.

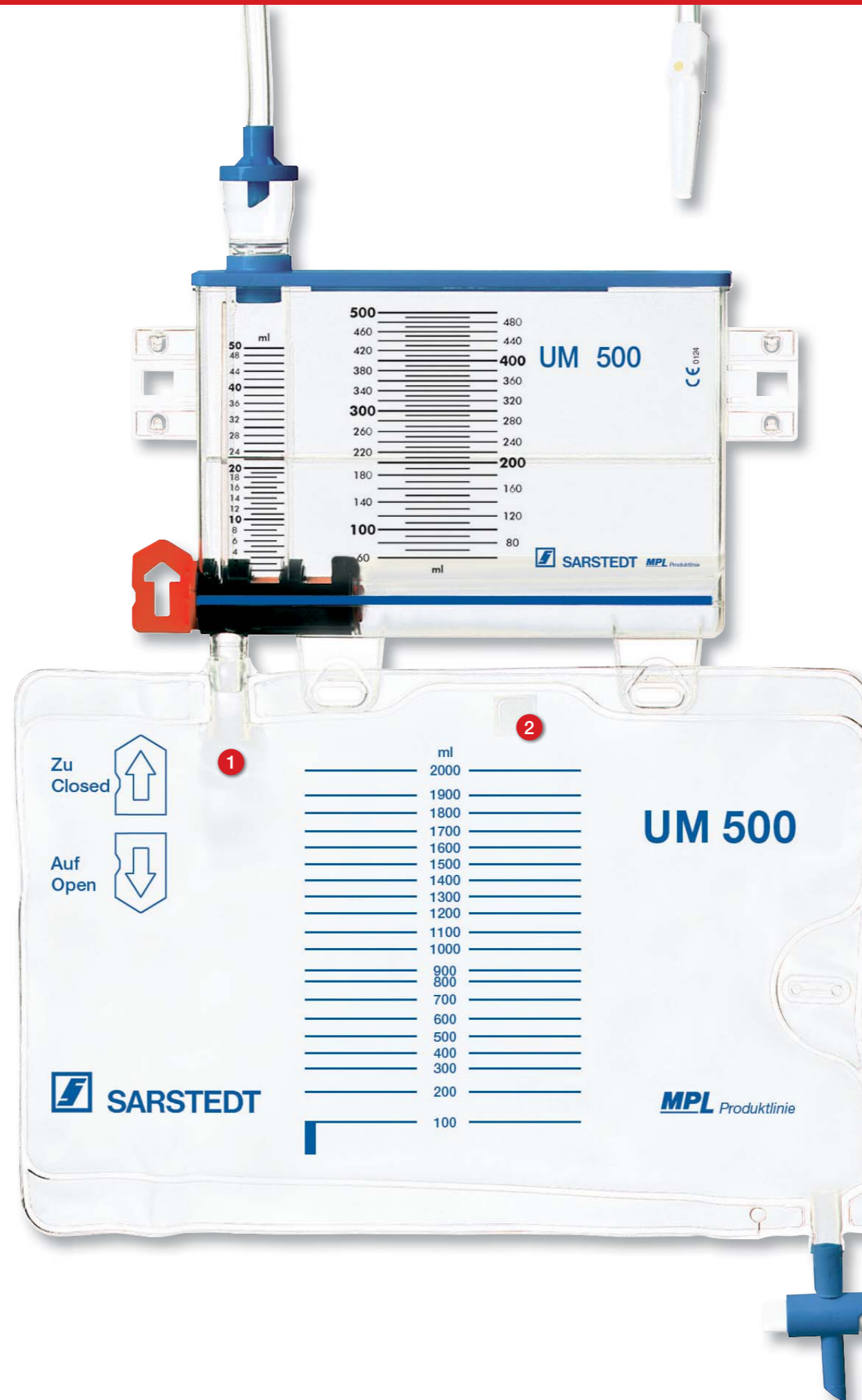Volumes exceeding 500 ml are automatically diverted through the integral overflow chamber and drained into the collection bag.

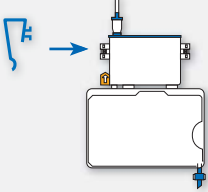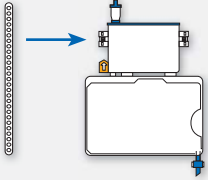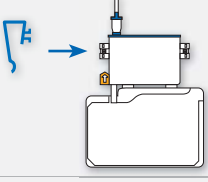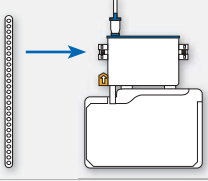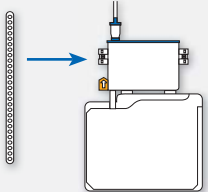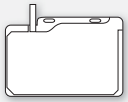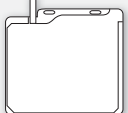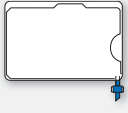

UM 500 Ordering Information

	Description	Order No.	Packaging Unit / Qty.
	UM 500 Urine measuring system with 2,000 ml drainage tap bag and side rail hooks <i>individually packed, sterile</i>	74.5610.100	15
	UM 500 Urine measuring system with 2,000 ml drainage tap bag and perforated elastic strap to fit side rails <i>individually packed, sterile</i>	74.5610.200	15
	UM 500 Urine measuring system with 1,800 ml exchange bag and side rail hooks <i>individually packed, sterile</i>	74.5620.100	15
	UM 500 Urine measuring system with 1,800 ml exchange bag and perforated elastic strap to fit side rails <i>individually packed, sterile</i>	74.5620.200	15
	UM 500 Urine measuring system with 3,000 ml exchange bag and perforated elastic strap to fit side rails <i>individually packed, sterile</i>	74.5630.200	15
	Exchange bag 1,800 ml <i>individually packed, sterile</i>	74.5622.005	100
	Exchange bag 3,000 ml <i>individually packed, sterile</i>	74.5624.005	100
	Exchange bag 2,000 ml, with drainage tap <i>individually packed, sterile</i>	74.5623.005	100
	Replacement strap	74.5763.010	50
	Additional hook for wide bed frames	74.5650.010	150

Technical modifications reserved

This document may contain information on products that may not be available in particular countries

254a-200-0610-1111

SARSTEDT AG & Co.
P.O. Box 12 20
D-51582 Nümbrecht
Phone +49 2293 3050
Fax +49 2293 305-122
info@sarstedt.com
www.sarstedt.com