

SMC 6 V

A powerful and variably adjustable bench centrifuge

NEW
Now with 2,500 x g




- 3 variably adjustable centrifuge programs
- Centrifugation up to 2,500 x g
- High quality and easy handling



3 applicable centrifugation programs

3 centrifugation programs are available.

- Program 1: Blood, 2500xg for 10 minutes
- Program 2: Urine, 500xg for 5 minutes
- Program 3: variably adjustable

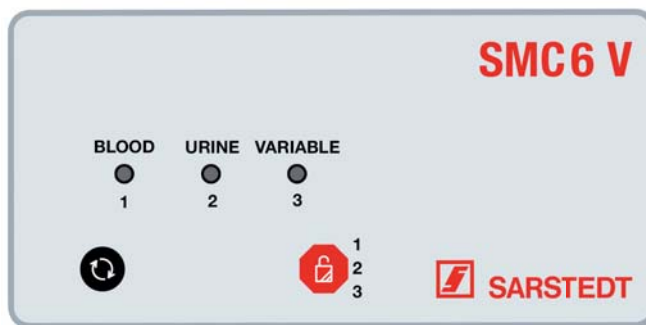

Optimum separating layer thanks to swing-out rotor


The SMC 6 V centrifuge is specially designed for use in medical practices or small laboratory units and can be used with all standard sample tubes. Therefore, a reliable gel barrier in serum or plasma tubes is immediately ensured after collecting blood. This diffusion barrier guarantees precise sample stability.

Centrifugation with swing-out rotor (SMC 6 V)

- Thick and stable separating layer
- More material available for analysers

Swinging bucket rotor


For optimal gel layer characteristics, we recommend that swing-out rotors be used for the centrifugation of the S-Monovette®.

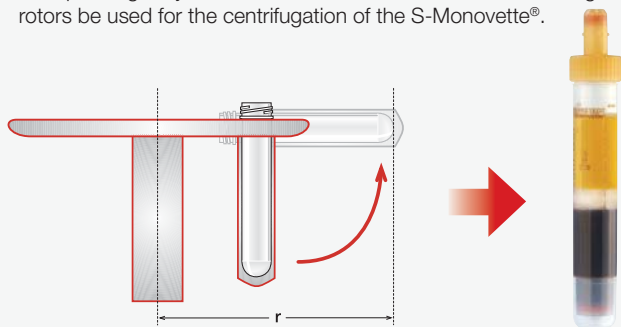

Centrifugation with fixed-angle rotor

- Thin and unstable separating layer
- Less material available for analysers

Fixed-angle rotor

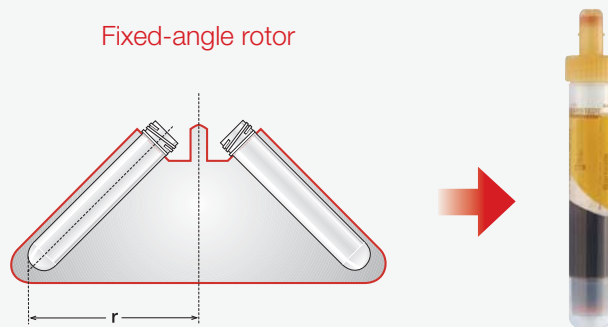

Easy to use

As operation is very simple using just two or three keys, operating errors and unintentional adjustment of speed and time can be avoided.

Ordering information

Order number	Description
90.184.400	SMC 6 V centrifuge

SARSTEDT AG & Co. KG
 P.O. Box 12 20
 D-51582 Nümbrecht
 Phone: +49 2293 305 0
 Fax: +49 2293 305 3992
 export@sarstedt.com
 www.sarstedt.com