

S-Sedivette®


Handhabung

1



Unmittelbar vor der Blutentnahme die S-Sedivette® mit einer Safety-Kanüle für die Blutentnahme mittels Aspirationstechnik komplettieren.

2

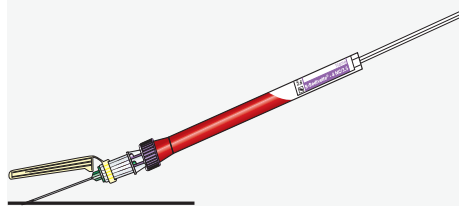


Führen Sie die Blutentnahme entsprechend Ihrer Arbeitsanweisung durch.

3




Nach der Blutentnahme den Kolben **unbedingt** einrasten und die Kolbenstange abbrechen!

4

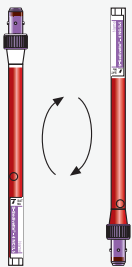

Die Blutprobe sofort gründlich mischen. Die S-Sedivette® fünfmal langsam über Kopf schwenken. Die Luftblase/Glasperle muss immer bis zum anderen Ende durchwandern! Kein schnelles Hin- und Herschütteln.

5

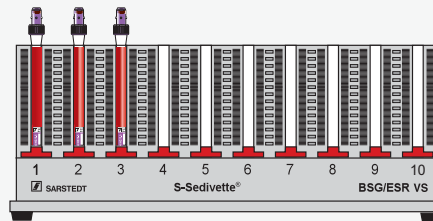


Die BSG-Bestimmung sollte innerhalb 4 Std. nach der Blutentnahme beendet sein. Die S-Sedivette® **vor Starten der Blutsenkung** noch einmal gründlich **mischen**. Dann den S-Sedivetten-Ständer bestücken und mit den Rändelschrauben (Pfeil) den Nullpunkt jeder Probe einstellen.

Hinweis für die Barcode-Etikettierung:




Etikett am Boden der S-Sedivette® ansetzen.

Richtig !

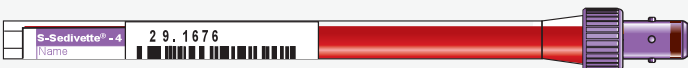

Falsch !

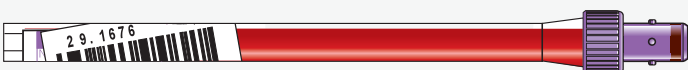

Falsch !

SARSTEDT AG & Co.
Postfach 12 20 · D-51582 Nümbrecht
Telefon (+49) 0 22 93 30 50
Telefax (+49) 0 22 93 305-282
☎ Service 0800 (Deutschland)
Telefon (0800) 0 83 30 50
info@sarstedt.com
www.sarstedt.com


 **SARSTEDT**

S-Sedivette®

User Guide


1

To collect blood using the aspiration method, attach S-Sedivette® to a Safety-Needle immediately before collection.

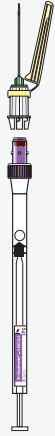

2

Proceed with blood collection in accordance with your internal instructions.

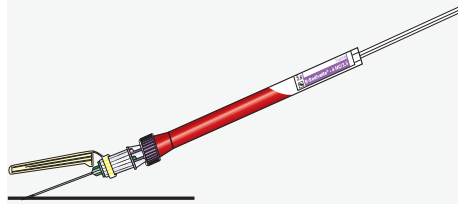


3

After blood collection, lock plunger into S-Sedivette® base and break off piston rod!




4

Mix sample thoroughly immediately after blood collection by inverting the S-Sedivette® 5 times upside down. Make sure that the air bubble/glass bead always moves from one end to the other. Do not shake rapidly.

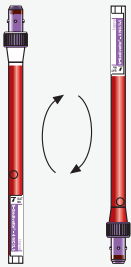

5

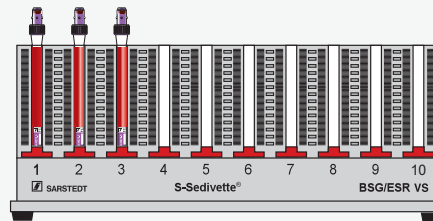


ESR determination should be carried out within a period of 4 hrs. after blood collection. Mix the S-Sedivette® thoroughly prior to ESR determination once again and place it into the S-Sedivette® rack. Use the knurled thumb screw (arrow) to adjust each sample to zero.

Barcode labelling instructions:




Important: Attach label at S-Sedivette® bottom.

Correct !

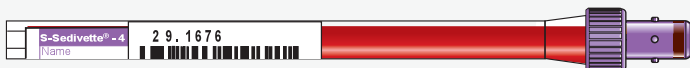

Wrong !

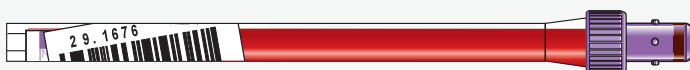

Wrong !

SARSTEDT AG & Co.
P.O. Box 12 20
D-51582 Nümbrecht
Phone +49 2293 305-0
Fax +49 2293 305-122
info@sarstedt.com
www.sarstedt.com

 **SARSTEDT**