



Seraplas® V-Filter


Ventilfilter zur Trennung von Serum und Blutkuchen nach Zentrifugation

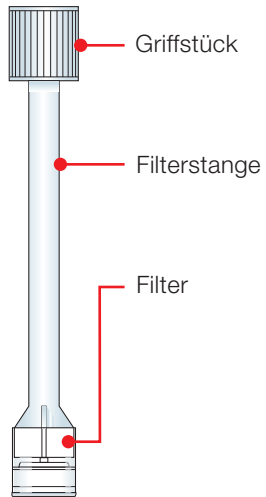

Falls Filterstange und Filter getrennt sind (z.B. durch Transport), bitte Filterstange in den längeren Schaft des Filtergehäuses eindrücken.

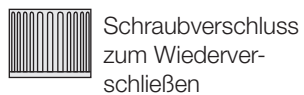


Schraubverschluss zum Wiederverschließen

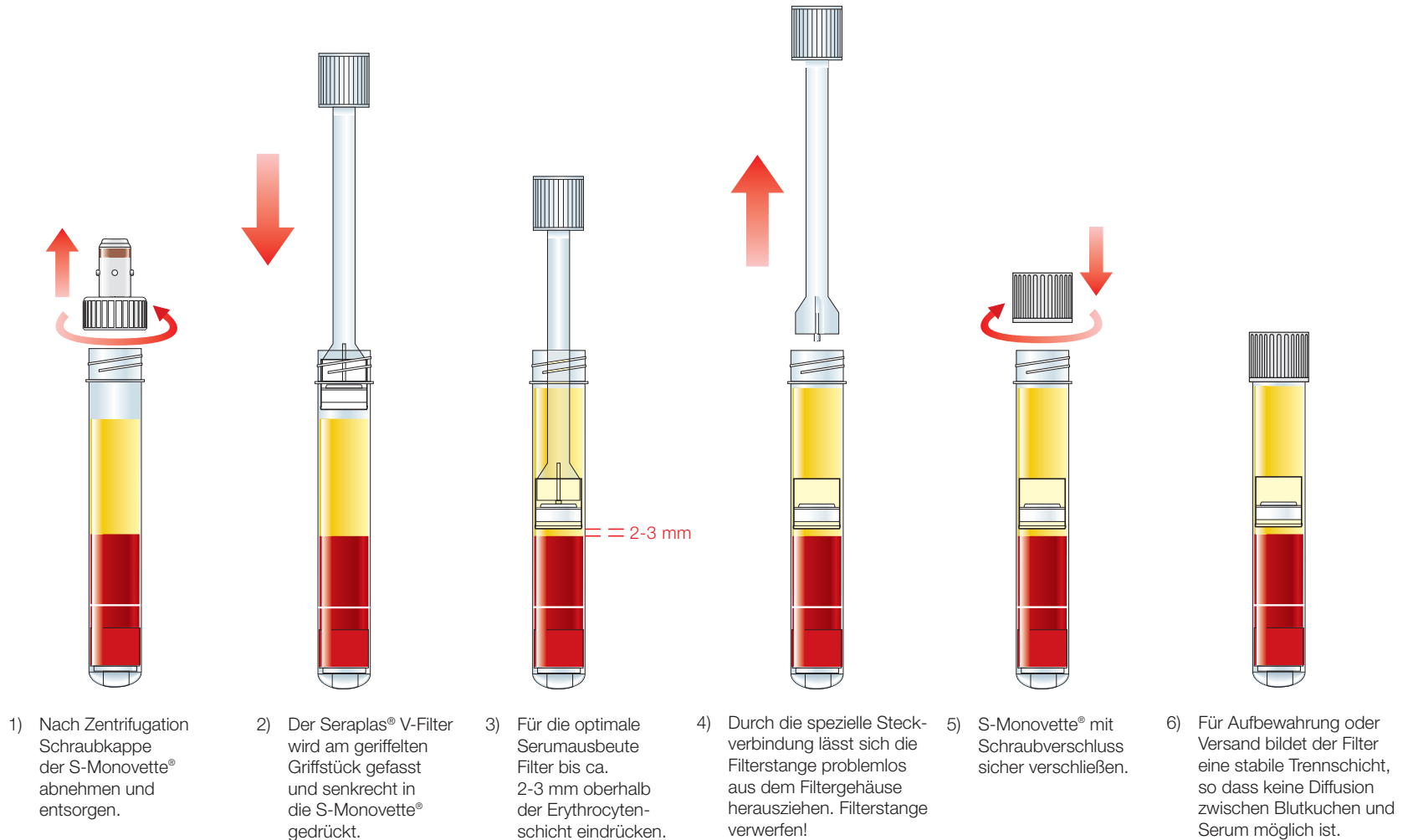


Seraplas® V-Filter

Valve filter for serum and blood clot separation after centrifugation

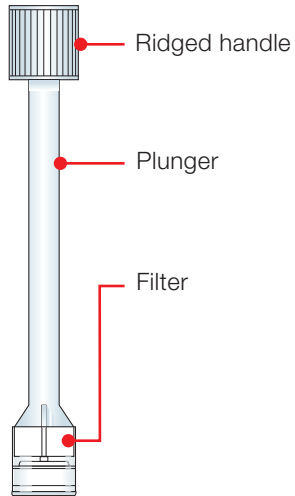

If the plunger and filter have detached (e.g. during transport), push the plunger into the extended shaft of the filter housing.

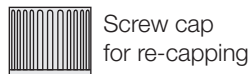

Screw cap for re-capping

-
- 1) Remove the screw cap from the S-Monovette® after centrifugation and discard.
 - 2) Hold the Seraplas® V-Filter by the ridged handle and insert it into the S-Monovette® in a vertical direction.
 - 3) For optimum serum separation, push the filter to approx. 2 to 3 mm above the erythrocyte layer.
 - 4) A special connection between the valve and plunger ensures easy withdrawal of the plunger from the filter housing. Discard the plunger.
 - 5) Attach screw cap for safe closure of the S-Monovette®.
 - 6) The filter forms a stable physical barrier preventing diffusion between the blood clot and serum for storage or transport.