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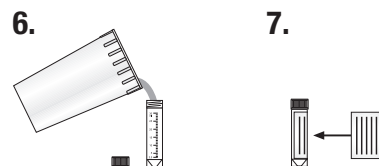
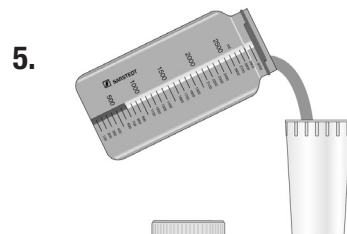
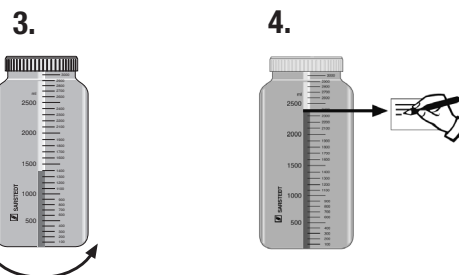
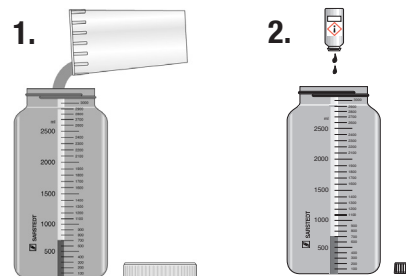
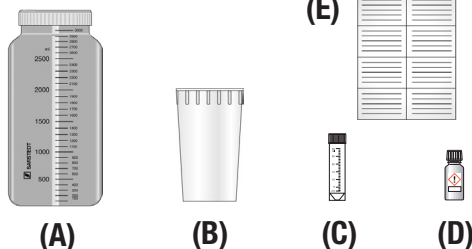
This is what you received:

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After getting up in the morning empty urinary bladder into the toilet. Write down the date and time on the enclosed label (E).

1. Please use the receptacle (B) for the following mictions and fill this urine into the urine container (A).
2. Then drip the complete contents (acid) from the small glass bottle (D) into the urine container (A).

Achtung
Warning
Attention

The small glass bottle contains 9 ml hydrochloric acid 20 %

Irritation to eyes, respiratory organs and skin. Keep tightly closed and inaccessible for children. In case of contact with eyes, flush with plenty of water immediately and consult a medical doctor. In case of accident or, if indisposed, seek medical advice immediately (if possible show this label).

3. Close the urine container (A) well and shake it. Collect any further urine, also in case of bowel movement, and fill it into the urine container (A). Close the urine container (A) well after every use and shake it. The next day, go to the toilet at the time noted down the day before and collect this urine. Collection is completed now.
4. Read the collected quantity from the scale on the urine container (A). **Write down this figure on the label (E).** Fill in the label (E) completely.
5. Fill some urine from the urine container (A) into the receptacle (B).
6. Fill the 30 ml tube (C) with urine to the 30 ml mark with urine. Empty the remaining urine from the receptacle (B) and the urine container (A) into the toilet.
7. Close the tube (C). Attach the completed label (E) onto the tube (C). Hand the filled tube (C) over to your physician. All components of the urine collection set can be disposed of as domestic waste.

(A): Urine Container (B): Receptacle (C): 30 ml tube
(D): Glass bottle contains 9 ml hydrochloric acid 20 % (E): Label for 30 ml tube
Store at room temperature • Technical modifications reserved

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SARSTEDT AG & Co.