


Safety-Multifly®

Instructions for 'Single-Handed Use'

When collection is complete,

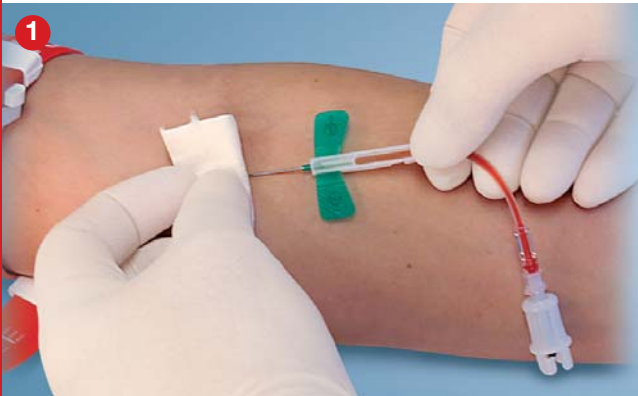

hold the needle protector at its back end with your thumb on top and your forefinger below and withdraw the Safety-Multifly® from the vein.



Affix the tubing by pressing it slightly against the palm of your hand...



...and push the needle protector over the needle...



...until the needle protector locks into place with a distinctly noticeable and audible "click".

IMPORTANT for single-handed use


▶ Affix the tubing by pressing it slightly against the palm of your hand! ◀



After activating the protective mechanism, discard the safely locked Safety-Multifly® needle in a disposal box.


Safety-Needle

User Guide

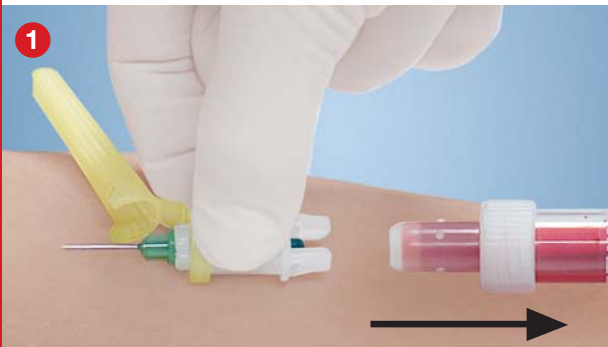

After blood collection

Remove the last S-Monovette® from the Safety-Needle and then withdraw the Safety-Needle from the vein.

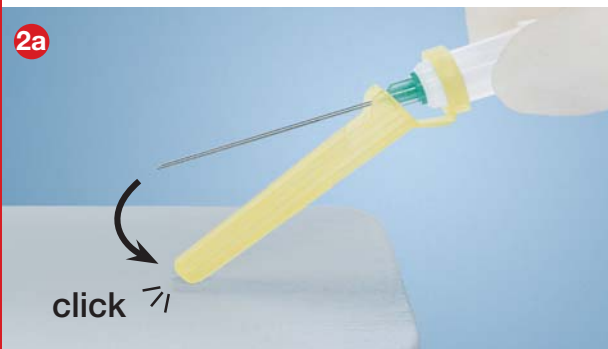

Hold the Safety-Needle at the adapter and, on a flat surface, push it downwards until the needle locks into the needle protector with a distinctly noticeable and audible "click".



Alternatively to Fig. 2a, you can also activate the needle protector using your index finger.

For safe operation, make sure to press against the lower end of the protection device.



After activating the protective mechanism

Discard the safely locked Safety-Needle in a disposal box.

SARSTEDT AG & Co.
P.O. Box 12 20
D-51582 Nümbrecht
Phone +49 2293 3050
Fax +49 2293 305-122
info@sarstedt.com
www.sarstedt.com